


# Owner's Instruction Manual

**Boccia**

**Mouthstick**

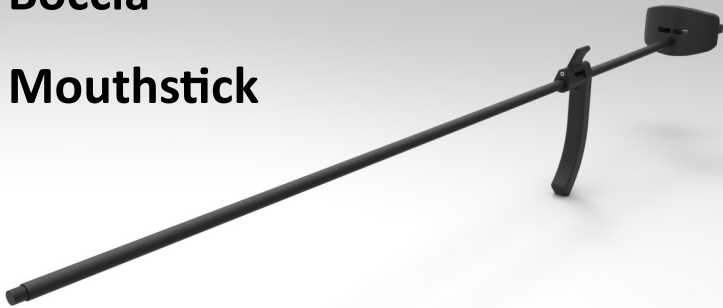

**WARNING**

READ THE OWNER'S MANUAL BEFORE USE.

**GRAVITY™**

*Premium Boccia Gear*

**TheraFin**  
Innovation

---

Thank you for purchasing The Boccia Mouthstick.

Please refer to these instructions for adjustment, care, and use of your product. Please note all warnings.





Caution: Read Instructions before operating device.

## Table of Contents

Parts Identification.....	3
Using the Mouthstick.....	4
Overall Dimensions .....	7
Care and Cleaning .....	8
Warranty Information .....	8

---


# Parts Identification

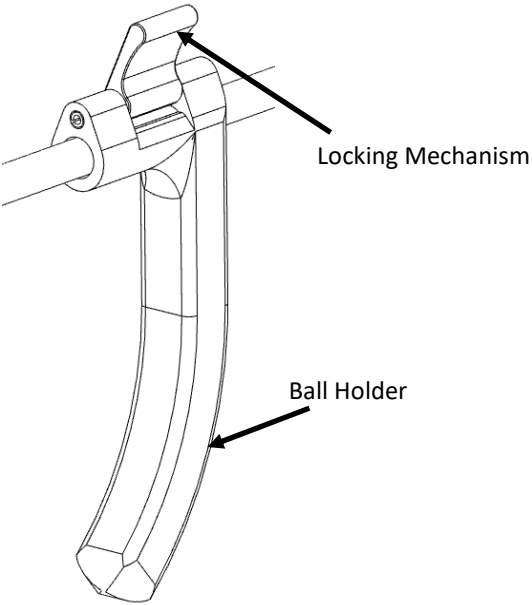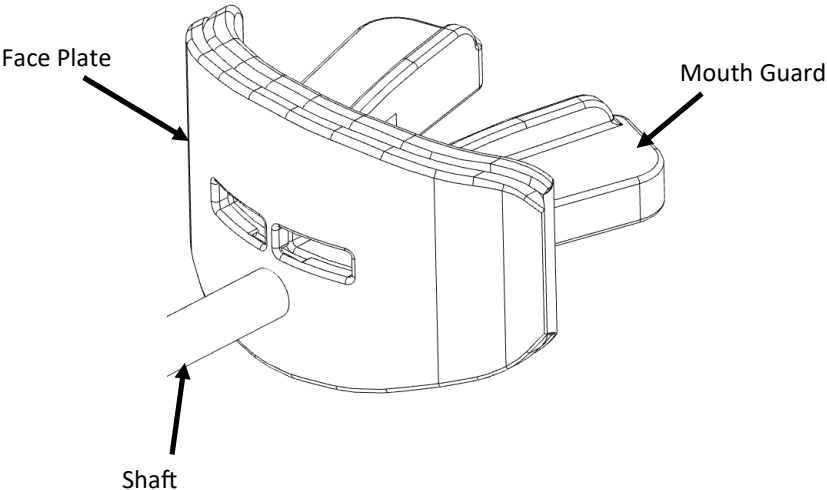

---

## Using The Mouthstick

Bite down on the mouth guard to secure it. Unhinge the locking mechanism to free it from the shaft. Adjust the ball holder until in the desired position. Lock the ball holder in place.

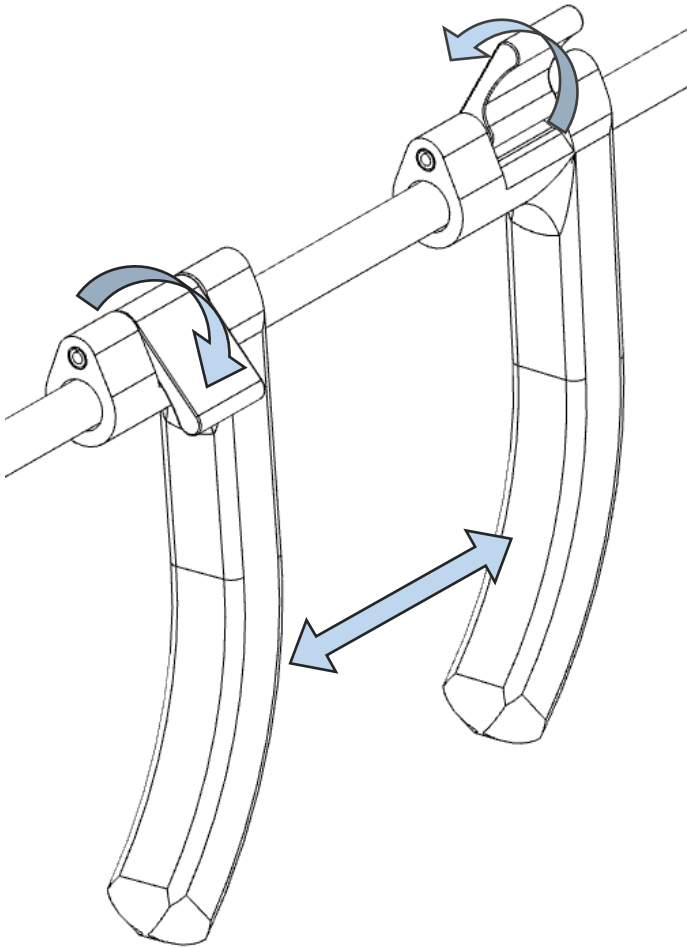


---

# Using The Mouthstick

Place the ball behind the ball holder. Release the ball by pulling the ball holder up.


For further information, visit [www.therafininnovation.com](http://www.therafininnovation.com).



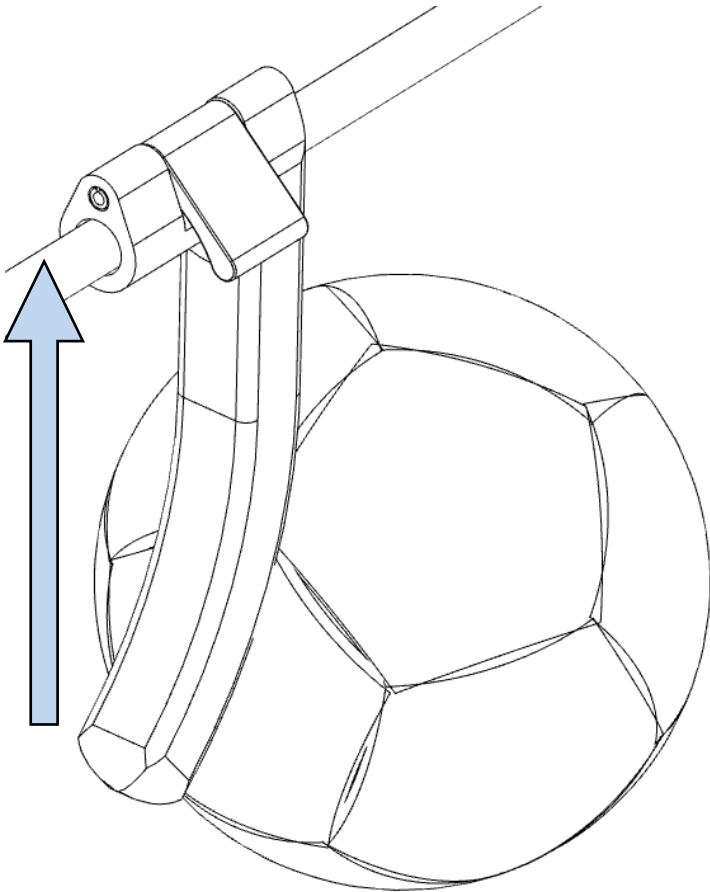


---

# Using The Mouthstick

The Mouthstick can also be used to push the ball over a wedge holder as shown in the figure below.

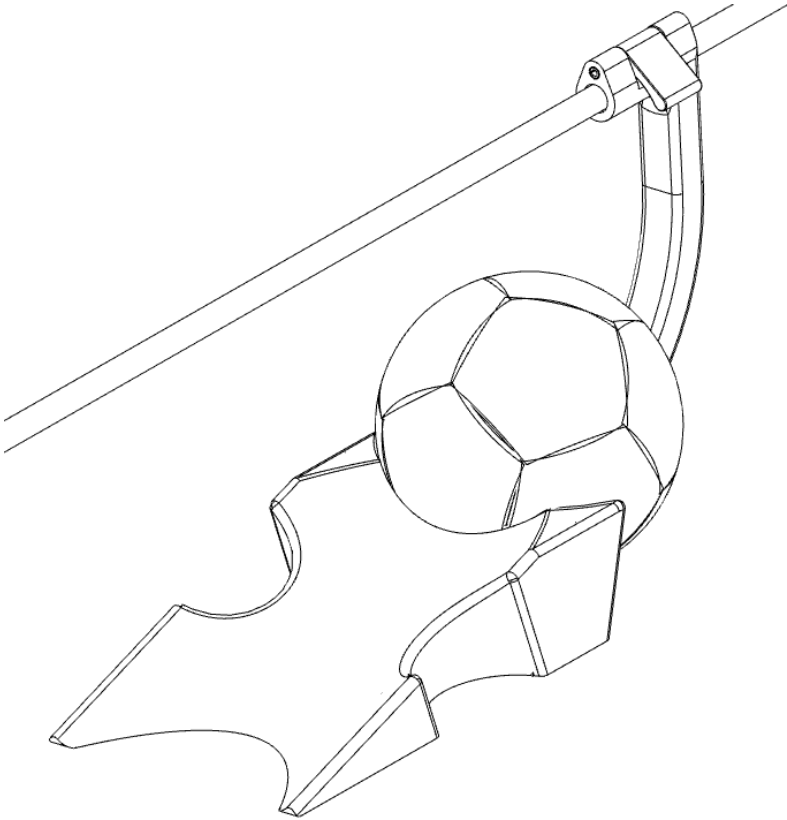

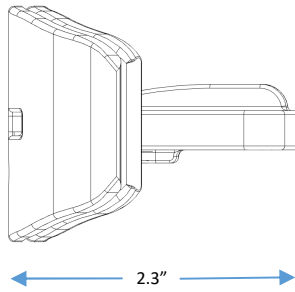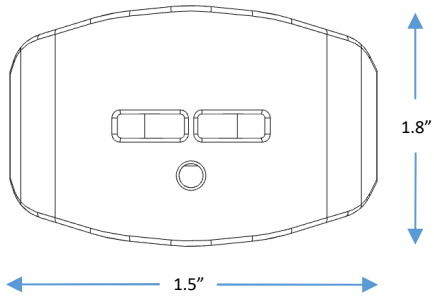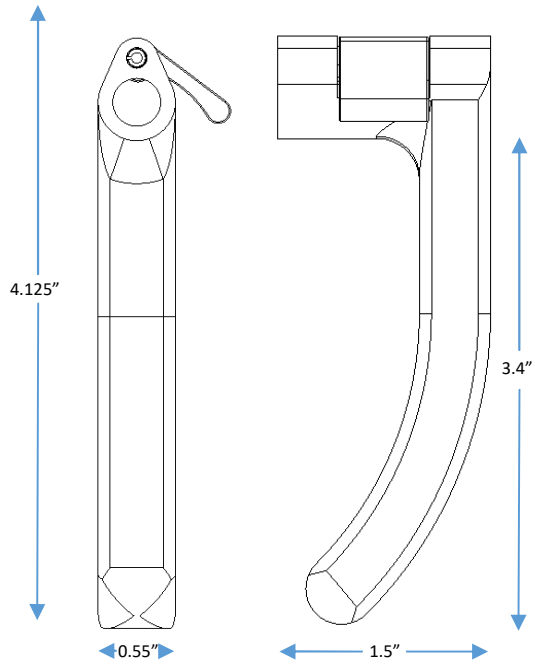

---

## Care and Cleaning

1. Keep your mouthguard clean and germ free by washing it with hand or dish soap and running water after each use.
2. You can also clean your mouthguard with toothpaste and a soft bristle toothbrush.
3. Do not leave your mouthguard in direct sunlight or in a hot car for an extended period of time-your mouthguard will melt!

*Document 08560  
Revision B 01-2018*

## Warranty

Any product found to be defective due to the material or workmanship within one year will be replaced under our return policy following product evaluation by Therafin Corporation. This replacement warranty does not include misuse or neglect of the product.



Therafin Corporation  
9450 W Laraway Road  
Frankfort, IL 60423   
800-843-7234

[www.therafininnovation.com](http://www.therafininnovation.com)